


Instuction For Use : Lancing Device

Front

Back


Lancing Device

The Lancing Device is used to obtain blood by using a Lancet. The adjustable cap on the Lancing Device allows you to control the depth of skin penetration and volume of blood needed. Before starting, wash your hands with warm soapy water and dry thoroughly. After completion, please follow the directions listed below.

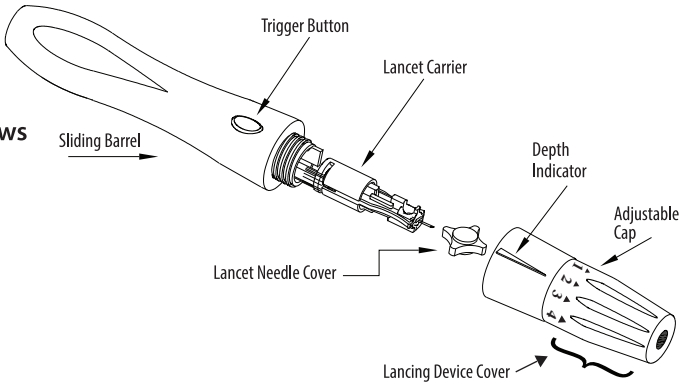


Directions

1. Remove the lancing device cap by rotating it counter-clockwise.
2. Insert a new Lancet firmly into the lancet carrier.
3. Secure the body of the lancet and twist the cover to remove.

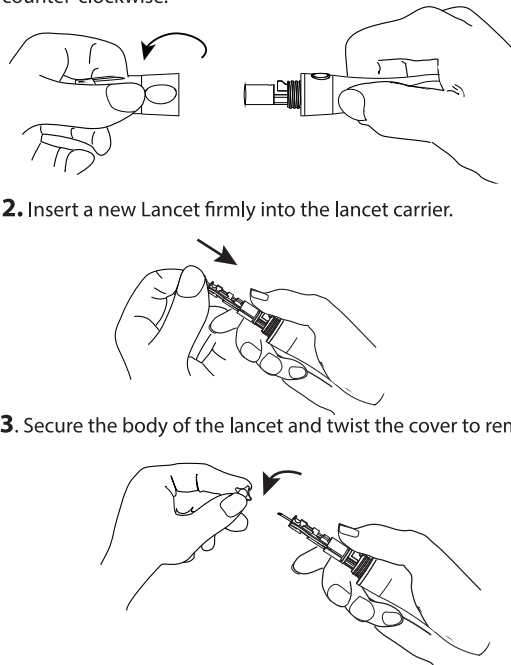


4. Place the lancing device cap back on the device and rotate clockwise to secure.
5. The adjustable cap offers 5 levels of skin penetration. To select a setting, use the "Skin Penetration Scale" displayed below to choose the one right for you.

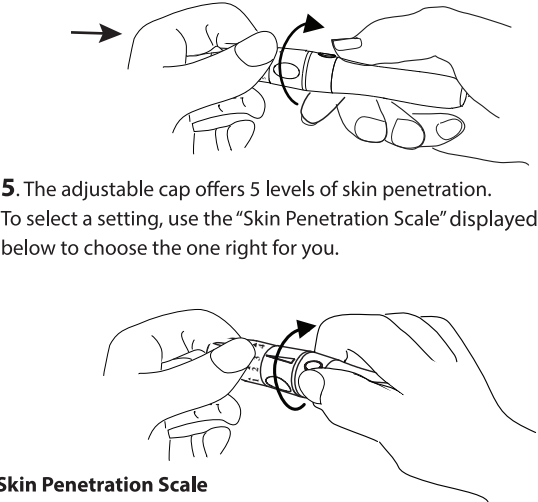


Skin Penetration Scale
1-2 (▲ ▲) for soft or thin skin
3 (▲) for average skin
4-5 (▲ ▲) for thick or calloused skin


6. Hold the lancing device in one hand and use the other hand to pull the handle back until a click is heard indicating that the lancing device is ready. Release the sliding handle to return it to its original position.

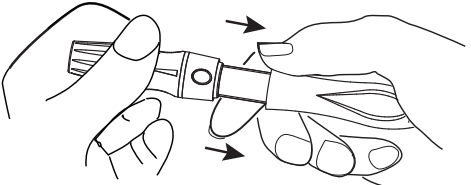

7. Position the lancing device cap firmly against the finger and/or desired area to be punctured, then press the button to fire.

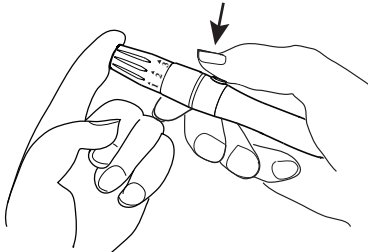


8. Remove the lancing device cap by repeating Step 1. Once the adjustable cap is removed, hold the body of the Lancet and firmly stick the lancet needle into the cover. Remove the lancet from the carrier and discard in an appropriate sharps or biohazard container.

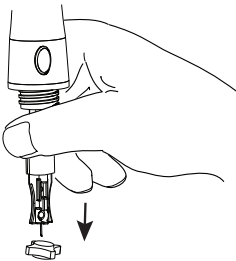

Alternate Site Testing (AST)

For Alternate Site Testing (AST), a clear cap has been included. Replace the adjustable cap with the clear cap, if AST is required.

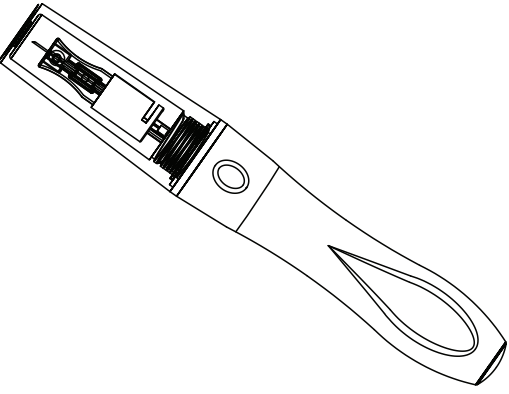

Taking Care of Your Lancing Device:

After use, remove the lancing device cap and wash using warm, soapy water. Rinse well and allow it to dry thoroughly. Use alcohol or mild soap and water to wipe the remaining outside of the Lancing Device.

Limited Warranty

The Lancing Device will function effectively for a period of six (6) months from the date of purchase. A malfunctioning device should be returned to your supplier for replacement.

CAUTION: For single patient use only.
Device is NOT intended for use on multiple individuals and/or patients!

Phoenix Healthcare Solutions, LLC.
Ft. Lauderdale, FL 33312
954-607-6080
www.phoenixhealthcaresolutions.com

EC REP

Mdi Europa GmbH,
Langenhagener Str.71
30855
Langenhagen, Germany

CE

Made in India